

Feces collection kit Pro-media FC-20



User-friendly, compact design

- The cap is easy to detach and reattach.**
The cap comes with fine grooves to prevent slippage when opening or closing it. It can therefore be easily detached and reattached with one hand. The cap is also compatible with smear devices.
- Roll prevention**
The body of the container features a hexagonal shape to prevent the container from rolling.
- Equipped with a teardrop-tipped applicator**
To easily collect a sample, simply insert the applicator into a stool (immediately after defecation) and pull it out. At inspection, a sample can be smeared directly on a medium.
- The bottle opening is designed to enable the easy removal of excess stool.**
The U-shaped opening allows excess stool to be easily wiped off at inspection.
- 1cm in diameter, 8cm in overall length**
Being small, this container requires little space and is not bulky during transport.

Place in a storage medium suited for transport

- The product uses **a modified Carry-Blair medium** ideally suited for storing samples.
- The **pH indicator** installed in the product clearly indicates whether a stool is contained within.

The Pro-media FC-20 is a new type of feces collecting container for sampling, storing, and transporting stools to be inspected for intestinal bacteria and other microbes.



SPECIFICATIONS		Formula per 1000ml	
Code No.	FC-20	Sodium thioglycolate	1.5g
Container material:	Polypropylene	Disodium phosphate	1.1g
Container dimensions:	φ 10mm×80mm	Sodium chloride	10.0g
Medium:	Modified Cary-Blair medium, 1ml	Calcium chloride	0.09g
Package:	300pcs (30pcs/fancy case)	Agar	3.0g
Storage condition:	Room temperature	pH indicator	0.012g
Expiration date:	1 year after manufacture		pH 8.4±0.2
Others:	Equipped with label for name of the patient		

◆ Specifications are subject to change without notice for remodeling or quality enhancement.

Manufacturer **ELMEX Limited**

2-2 Ichigayasadohara-cho, Shinjuku-ku, Tokyo 162-0842 JAPAN
TEL : 081-3-5229-7911 FAX : 081-3-5261-4130
URL : <http://www.elmex.co.jp> e-mail address : sales@elmex.co.jp